

**Bonding Agent
Instructions**

Etch the enamel surrounding the cavity.

Wash and dry. Apply bonding agent and cure for 10 seconds. Bonding agent sets leaving a very thin layer of uncured liquid on the surface, which should not be wiped off. This layer will provide better adhesion to the restorative applied thereafter. Place restorative material with tapping movement and cure.

Dent Zar, Inc.
19643 Trull Brook Dr.
Tarzana, CA 91356
U.S.A.
Head Office
800-444-1241
323-465-3621
Fax: 323-465-2994

DENT ZAR, INC.
EC Authorized Representative
Mr. R.J. Macmillan
24-28 St Leonards Road #82
Windsor, Berkshire SL4 3BB
United Kingdom
0-500-892-711
or
0-800-960-750

125 A 1030 Denman St #304
Vancouver B.C. V6G 2M6
Canada
800-444-1241

Suite V15, 9 Crofts Avenue
Hurstville, NSW 2220
Sydney
Australia
1-800-152583

Suite 260,
453a Mt Eden Road
Mount Eden
Auckland 1024
New Zealand
0-800-55-9635

2A Crawford Hall
Wandesford Quay
Western Rd., Cork
Ireland
1800-559-997